

CREATING THE PERFECT FOAM BOX IMPRESSION

This is a step by step guide on how to create the perfect impression of the foot using a foam box. It is so important your impressions are created giving the most details so our technicians can manufacture your orders perfectly.

PREPARING THE PATIENT

Open your impression foam box.

Ensure your patient is standing or sitting over the top of the foam with their feet either side of the box.



CREATING THE IMPRESSION

Place one foot into one side of the foam box and push the entire foot into the foam. Ensure the entire foot, including the toes, have been pushed into the foam. Gently remove the foot and repeat for the other foot.

Fill in your prescription form and send this into us with the foam box, or email it us directly.

POST IT TO US

You can send your foam boxes to: Blueprint Orthotics, Unit F Coney Green Network Centre, Wingfield View, Claycross, Chesterfield, S45 9HX. You can also email your prescription forms to info@blueprintorthotics.com

